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Breath is Life

we can help you

MPVTRUMA

Calimero[®] plus SIDS-Monitor
zur Atmungs- und Herzüberwachung
for breathing and heart monitoring

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Calimero®plus advantages for the physician

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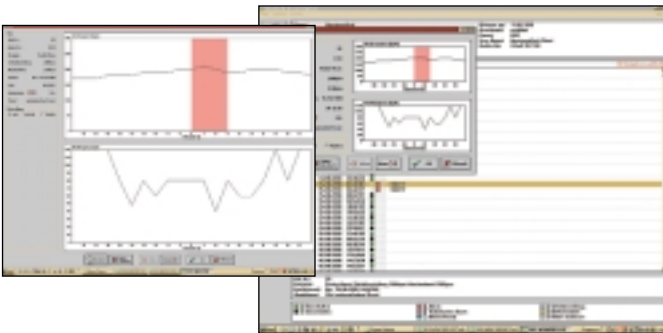
Intelligent logging function

Calimero®plus stores all events down to the nearest second.

The logging function, which is always active, records the following events: "On/Off, reset, all types of alarms, heart and breathing frequency before and after each alarm". The values can later be retrieved from a PC.

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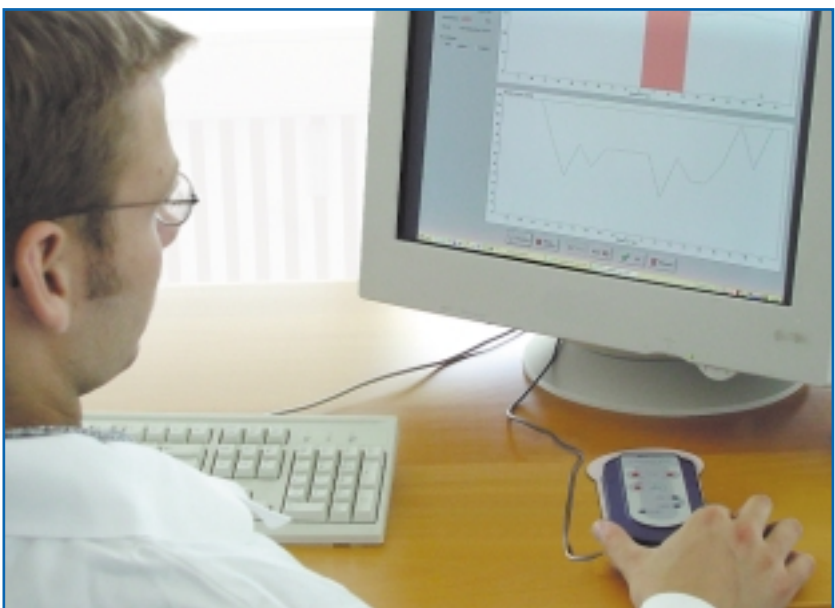
data transfer

Calimero®plus / Calimero®plus A offers an intelligent data transfer with the innovative read out pad alternative for use with Laptop or PC.

Simply place Calimero®plus / Calimero®plus A on the read out pad. Start the Calimero®plus / Calimero®plus A data analysis software and press the relevant button combination. The data will be transferred thanks to a new patented transponder technology (without cable).

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Security wherever you go - even if you are next door

Calimero®plus gives a clearly audible alarm if measured values are exceeded or fall below set values, which are easy to adjust with a screwdriver. With the external signaller and 15 m of cable the monitoring can securely take place from another room. A light diode shows regular breathing. In case of an alarm an optical and additionally an acoustic signal are triggered.

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Be prepared and help!

The timely alarm in case of irregularities allows for lifesaving measures within a minimum period of time. These measures should be trained beforehand in special courses. Enclosed training video „wohlbehütet“ shows the most important situations parents should be prepared for.

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Respiration bag supports first aid

The VentCare respiration bag is usually prescribed with the device and supports first aid measures if required.

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Non-intrusive monitoring

Calimero®plus operates without mains and using only long-life batteries. The method of monitoring does not require the continuous flow of a measurement current through the body - it is therefore safe and **unsurpassably gentle to the child.**

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NEW! Smallest monitor in the world

Thanks to state-of-the-art electronics, its comfortable, miniature construction makes it decidedly easy to use and completely inconspicuous. It is the smallest of its kind and doesn't look like an "intensive care unit".

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SIDS stands for **Sudden Infant Death Syndrome**, a phenomenon which even to this day claims the lives of **1 in every 500 infants**.

80% of infants affected by it are fully healthy beforehand. Only 20% belong to so-called risk groups (former premature babies or infants with a previous life threatening event or a SIDS death occurring with a close relative).

A SIDS death can as a rule be attributed to the sudden onset of **respiratory arrest** from a variety of causes, accompanied by cardiac dysrhythmia. The cardiac dysrhythmia itself can however be the primary cause of a SIDS death.

The **Calimero®plus** SIDS Monitor gives you the **security of reliable monitoring**, even while the infant is not under direct observation.

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Universal application

Whether in a clinic, at home or out and about in the pram or car. Calimero®plus is easy to transport (only 260 g) and reliably raises an alarm if worst comes to worst.

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MPVTRUMA

... for reliable breathing and heart monitoring

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Trouble-free operation

The device continuously monitors breathing frequency by means of a body sensor and heart frequency by means of two electrodes.

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Reliable breathing and heart monitoring

Calimero®plus can distinguish longer breathing pauses from a regular breathing rhythm and **raises an alarm** when a critical point, which can be tailored to the individual, has been reached.

It monitors heart function at the same time.

When measured values exceed or fall below set values, again an alarm is raised.

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Automatic checking

Calimero®plus is a micro-processor controlled self-monitoring device that automatically raises an alarm or acoustic signal if proper functioning or safety is impaired.

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Simple to use

Simply attach the body sensor and two single-use electrodes - a press of a button reliably initiates monitoring.

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Home monitoring avoids the need for hospital stays and offers reliable SIDS prevention in cases of increased risk.

Important prerequisites are:

1. Infant belongs to a risk group
2. Parents are fully committed to the monitoring
3. Non-stop monitoring of the device must be guaranteed

Period of monitoring:

The monitoring period can vary between 6 to 12 months depending on the risk assessment for the individual as well as practical and psychological aspects!

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A German medical prescription form (Rezept) for Calimero®plus SIDS-Monitor. The form is filled out with the following information:
- Patient name: Huchmann, Max
- Date: 23.2.22
- Prescriber: 42345, 42345667
- Dispenser: 42345, 230102
- Product name: Calimero®plus SIDS-Monitor
- Indication: Ind. Herzrhythmusstörungen
- Signature: [Handwritten signature]

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The Calimero®plus SIDS Monitor was specially developed for economical monitoring. We have refrained from using recording functions that are more often than not superfluous and lead to frequent false alarms (e.g. pulsoximetry).

Calimero®plus is unsurpassably economical in every detail:

1. Standard battery operation

A standard 9 volt battery with a lifespan of 1.5 - 3 months can be easily and economically replaced by the parents. There is also an option to use a special long-life battery. The device provides adequate warning when the battery needs replacing.

2. Operating and learning advantages

The device is very easy for parents to learn to use. It requires minimum servicing.

3. Reusable body sensor

The body sensor is durable, reusable and therefore saves money.

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4. Hardly any or no expendable parts

Expendable single-use electrodes can be replaced by a chest strap with long-life electrodes.

A real alternative in case of an allergic reaction to single-use electrodes.

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Quality you can count on

These advantages make Calimero®plus the best value for money SIDS monitor available contributing significantly to reducing costs!

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Technical Data

Operation mode:	Battery-operated, 9V monobloc battery, Battery type: alkaline or lithium
Battery replacement:	after approx. 1,5 or 3 months if operated with lithium battery
Operation temperature:	10 - 40° C
Storage temperature:	4 - 50° C
Connections:	Calimero®plus: Patient main cable, optionally external signal transmitter hospital pager or alarm center Calimero®plus A: optionally external signal transmitter hospital pager or alarm center
Alarm delay times:	10, 15, 20, 25 sec. (respiration)
Min./max. values:	Calimero®plus: 60, 70, 80, 90 / 180, 200, 220, 240 BPM (heart rate)
Dimensions:	L x w x h (in mm): 152 x 80 x 29
Weight:	approx. 230 g
Protection class:	BF
Classification to MPG:	IIB, 93/42/EEC
Mark of conformity:	CE 0125 Calimero®plus/Calimero®plus A conforms to the requirements of the 93/43EEC guideline and the German Medical Products Act (MPA) and has been EU tested.
Manufacturer:	Schulte Elektronik, Olsberg
We reserve the right to make any alternations!	

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Calimero®plus A Reliable respiration monitoring (without cardiac function)

The respiration rate is permanently monitored by a body sensor.
Calimero®plus A SIDS-Monitor can distinguish extended respiratory intervals from regular breathing and triggers an alarm when a critical value is reached.

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